



CERTIFICATION by CUSTODIAN of RECORDS  
Tucson Police Department Crime Laboratory

134

I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 5000 Serial Number 64-002659 (#13) maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R9-14-404.A.3, for a 0.080AC solution are 0.070AC to 0.090AC.

Date: 3/23/06 Cal 1 0.077 AC Cal 2 0.080 AC

Date: 4/20/06 Cal 1 0.079 AC Cal 2 0.081 AC

Date: N/A Cal 1 \_\_\_\_\_ AC Cal 2 \_\_\_\_\_ AC

Between the dates of 3/23/06 and 4/20/06 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on N/A.


The instrument was returned to service on N/A.

Jennifer L. Coombs  
Jennifer L. Coombs

4/21/06

ADPS QAS Permit # QAS 29886

SUBSCRIBED AND SWORN to before me this 21, day of April, 2006.

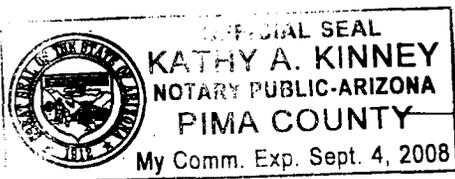

Kathy A. Kinney  
Notary Public

\*Includes documents specified in A.A.C. R9-14-404.A.6

EXHIBIT Q

1 of 6  
file

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 5000\*

AM

AGENCY Tucson Police QA SPECIALIST Jennifer Combs 46160  
(Print Name)

INTOXILYZER SERIAL # 64-002659 (#13) LOCATION DUI #10

DATE April 20 19 2006 TIME 1449

Display reads "READY TO START" or "PUSH BUTTON TO START TEST."

DIAGNOSTIC TESTS

- DVM Test check. Setting should be between .200 and .600. Mode selection switch S2 on, S1 and S3 off or keyboard menu selection "H". Reading is .376.
- Display Test check. Mode selection switch S1 on and S2 and S3 off or keyboard menu selection "V".
- Printer Test check. Mode selection switch S1, S2, S3 off or keyboard menu selection "P".
- Clock time check. Mode selection switch S10 on or keyboard menu selection "E".
- Date check. Mode selection switch S11 on or keyboard menu selection "E".

OPERATIONAL TESTS

- Alcohol free subject Test result 0.000 AC
  - Error recognition logic system functioning  
Invalid test printed
  - Proper sample recognition system  
Invalid sample printed ✓  
Deficient sample printed ✓
  - N/A Completeness of sample purge with collection tube\*\*
  - Calibration standard 0.080 AC  
Results: 0.079 AC & 0.081 AC
- Instrument operating properly and accurately. YES  NO

COMMENTS:

\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE

Jennifer L. Combs

0. N/A alcohol concentration standard collected for subsequent analysis.\*\*

**\*WITH OR WITHOUT VAPOR RECIRCULATION AND WITH OR WITHOUT KEYBOARD  
\*\*THIS STEP IS ONLY REQUIRED IF THIS DEVICE IS BEING USED FOR  
SAMPLE CAPTURE.**

DHS/DSLS/Form C117 (Rev. 7-93)

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.



# Tucson Police Department Crime Laboratory By

3 of 6  
gfi

#13

FEDERAL SIGNAL CORP.      TMI INC

INTOXILYZER      AIRBATH      ANALYZER

AZ MODEL 5000      SN 64-002659

04/20/06

TEST	BAC VALUE	TIME
AIR BLANK	.000	15:04
CAL. CHECK	.081	15:04
AIR BLANK	.000	15:05

TIME FIRST OBSERVED \_\_\_\_\_

SUBJECT'S NAME DUI HQ

INSTRUMENT LOCATION \_\_\_\_\_

OPERATOR 46160

ADDITIONAL INFORMATION AND/OR REMARKS \_\_\_\_\_

INTOXILYZER® INSTRUMENT PRINTER CARD

© 1986 by CMI INC.

#13

FEDERAL SIGNAL CORP.      CAL LAB

INTOXILYZER      ALCOHOL ANALYZER

AZ MODEL 5000      SN 64-002659

04/20/06

TEST	BAC VALUE	TIME
AIR BLANK	.000	15:02
CAL. CHECK	.079	15:03
AIR BLANK	.000	15:03

TIME FIRST OBSERVED \_\_\_\_\_

SUBJECT'S NAME DUI HQ

INSTRUMENT LOCATION \_\_\_\_\_

OPERATOR 46160

ADDITIONAL INFORMATION AND/OR REMARKS \_\_\_\_\_

INTOXILYZER® INSTRUMENT PRINTER CARD

© 1986 by CMI INC.

By 4 of 6  
JH

SN 64-002659 04-20-06 #13  
E47.02 10:56  
INVALID TEST  
SAMPLE INTRODUCED  
AT IMPROPER TIME.

---

---

SUBJECT'S NAME

---

---

TIME FIRST OBSERVED INSTRUMENT LOCATION  
DUI HQ

---

---

OPERATOR  
46160

---

---

ADDITIONAL INFORMATION AND/OR REMARKS

---

---

---

INTOXILYZER INSTRUMENT PRINTER CARD **EMI**<sub>INC</sub>  
© 1986 by CMI INC

5 of 6  
file

EXHIBIT P

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 5000\*

24

STANDARD CALIBRATION CHECK PROCEDURE

Agency Tucson Police Department Date 4/20/00 Time 1449  
Intoxilyzer Serial # 14-002659 (#13) Location DLU 110  
QA Specialist Jennifer Combs 46160  
(Print Name)

- ( ✓ ) 1. Pour a standard alcohol solution of known value into a clean dry simulator jar and assemble the simulator. Insure that a tight seal has been made. Standard value: 0.080 AC
- ( ✓ ) 2. Turn on the simulator and allow the temperature to reach 34°C ± .2°C.
- ( ✓ ) 3. Set Intoxilyzer mode selection in the ACA mode by switching mode selection switch #9 on or selecting "C" on keyboard menu.
- ( ✓ ) 4. Attach simulator to the simulator entrance port on the Intoxilyzer.
- ✓ ✓ 5. Intoxilyzer 5000 display reads "READY TO START" or "PUSH BUTTON".
- ✓ ✓ 6. Push Start Test button or press enter on keyboard.
- ✓ ✓ 7. Insert card in response to display.
- ✓ ✓ 8. Air blank completed.
- ✓ ✓ 9. Calibration check completed. Test results 0.079 AC & 0.081 AC
- ✓ ✓ 10. Air blank completed.
- ✓ ✓ 11. When display reads Test Complete remove evidence card. Attach the card to the completed checklist.
- ( ✓ ) 12. Return mode selection switch #9 to off position after all calibration checks are complete or type Q and enter on keyboard.

SIGNATURE Jennifer L. Combs

\*WITH OR WITHOUT VAPOR RECIRCULATION AND WITH OR WITHOUT KEYBOARD

DHS/DSLS/Form C116 (Rev. 7-93)

#13

FEDERAL SIGNAL CORP. / CMI INC  
INTOXILYZER - ALCOHOL ANALYZER  
AZ MODEL 5000 SN 64-002659  
04/20/06

TEST	BAC VALUE	TIME
AIR BLANK	.000	14:57
INVALID SAMPLE	.000	14:57
AIR BLANK	.000	14:57

SUBJECT'S NAME

TIME FIRST OBSERVED

DULL HQ  
INSTRUMENT LOCATION

46160  
OPERATOR

ADDITIONAL INFORMATION AND/OR REMARKS

Sweet Breath

INTOXILYZER INSTRUMENT PRINTER CARD

© 1986 by CMI INC



#13

FEDERAL SIGNAL CORP. / CMI INC  
INTOXILYZER - ALCOHOL ANALYZER  
AZ MODEL 5000 SN 64-002659  
04/20/06

TEST	BAC VALUE	TIME
AIR BLANK	.000	14:58
*SUBJECT TEST	.078	15:00
AIR BLANK	.000	15:01

\* DEFICIENT SAMPLE - VALUE  
PRINTED WAS HIGHEST OBTAINED.

SUBJECT'S NAME

TIME FIRST OBSERVED

DULL HQ  
INSTRUMENT LOCATION

46160  
OPERATOR

ADDITIONAL INFORMATION AND/OR REMARKS

INTOXILYZER INSTRUMENT PRINTER CARD

© 1986 by CMI INC



6 of 6  
file

39