



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory

BM

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 5000 Serial Number 164-003336 (#15) maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R9-14-404.A.3, for a 0.080AC solution are 0.070AC to 0.090AC.

Date: 3/23/06 Cal 1 0.078 AC Cal 2 0.079 AC

Date: 4/20/06 Cal 1 0.080 AC Cal 2 0.080 AC

Date: N/A Cal 1 AC Cal 2 AC

Between the dates of 3/23/06 and 4/20/06 the instrument was operating properly and accurately.


The instrument was taken out of service for repair on 4/20/06. The instrument was returned to service on N/A.

Jennifer L. Coombs
Jennifer L. Coombs

5/2/06

ADPS QAS Permit # QAS 29886

SUBSCRIBED AND SWORN to before me this 2, day of May, 2006.

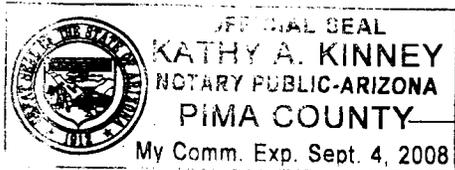

Kathy A. Kinney
Notary Public

*Includes documents specified in A.A.C. R9-14-404.A.6

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

EXHIBIT Q

1 of 6
file

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 5000*

SM

AGENCY Tucson Police QA SPECIALIST Jennifer Combs 46116
(Print Name)

INTOXILYZER SERIAL # 104-003356(115) LOCATION Dell HQ
DATE April 20 19 2016 TIME 1511

Display reads "READY TO START" or "PUSH BUTTON TO START TEST."

DIAGNOSTIC TESTS

- DVM Test check. Setting should be between .200 and .600. Mode selection switch S2 on, S1 and S3 off or keyboard menu selection "H". Reading is .205.
- Display Test check. Mode selection switch S1 on and S2 and S3 off or keyboard menu selection "V".
- Printer Test check. Mode selection switch S1, S2, S3 off or keyboard menu selection "P".
- Clock time check. Mode selection switch S10 on or keyboard menu selection "E".
- Date check. Mode selection switch S11 on or keyboard menu selection "E".

OPERATIONAL TESTS

- Alcohol free subject Test result 0.000 AC
 - Error recognition logic system functioning
Invalid test printed
 - Proper sample recognition system
Invalid sample printed ✓
Deficient sample printed ✓
 - N/A Completeness of sample purge with collection tube**
 - Calibration standard 0.150 AC
Results: 0.080 AC & 0.050 AC
- Instrument operating properly and accurately. YES NO

COMMENTS: _____

SIGNATURE Jennifer T. Combs

0. N/A alcohol concentration standard collected for subsequent analysis.**

*WITH OR WITHOUT VAPOR RECIRCULATION AND WITH OR WITHOUT KEYBOARD
**THIS STEP IS ONLY REQUIRED IF THIS DEVICE IS BEING USED FOR
SAMPLE CAPTURE.

DHS/DSLS/Form C117 (Rev. 7-93)

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

Tucson Police Department Crime Laboratory

BH
3 of 6
file

#15

CMI INC
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 64-003336
04/20/06

TEST	BAC VALUE	TIME
AIR BLANK	.000	15:12
CAL. CHECK	.000	15:12
AIR BLANK	.000	15:13

DVM = .205

SUBJECT'S NAME _____

INSTRUMENT LOCATION DUI HQ

TIME FIRST OBSERVED _____

OPERATOR 46160

ADDITIONAL INFORMATION AND/OR REMARKS _____

INTOXILYZER INSTRUMENT PRINTER CARD

© 1986 by CMI INC



#15

CMI INC
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 64-003336
04/20/06

TEST	BAC VALUE	TIME
AIR BLANK	.000	15:11
CAL. CHECK	.000	15:11
AIR BLANK	.000	15:11

SUBJECT'S NAME _____

INSTRUMENT LOCATION DUI HQ

TIME FIRST OBSERVED _____

OPERATOR 46160

ADDITIONAL INFORMATION AND/OR REMARKS _____

INTOXILYZER INSTRUMENT PRINTER CARD

© 1986 by CMI INC



EXHIBIT P

5 of 6
jlc

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 5000*

AM

STANDARD CALIBRATION CHECK PROCEDURE

Agency Tucson Police Department Date 4/20/00 Time 1511
Intoxilyzer Serial # 64-003336 (#15) Location DUI 116
QA Specialist Jennifer Combs 46/100
(Print Name)

- (✓) 1. Pour a standard alcohol solution of known value into a clean dry simulator jar and assemble the simulator. Insure that a tight seal has been made. Standard value: 0.180 AC
- (✓) 2. Turn on the simulator and allow the temperature to reach 34°C±.2°C.
- (✓) 3. Set Intoxilyzer mode selection in the ACA mode by switching mode selection switch #9 on or selecting "C" on keyboard menu.
- (✓) 4. Attach simulator to the simulator entrance port on the Intoxilyzer.
- ✓ (✓) 5. Intoxilyzer 5000 display reads "READY TO START" or "PUSH BUTTON".
- ✓ (✓) 6. Push Start Test button or press enter on keyboard.
- ✓ (✓) 7. Insert card in response to display.
- ✓ (✓) 8. Air blank completed.
- ✓ (✓) 9. Calibration check completed. Test results 0.080 AC & 0.080 AC
- ✓ (✓) 10. Air blank completed.
- ✓ (✓) 11. When display reads Test Complete remove evidence card. Attach the card to the completed checklist.
- (✓) 12. Return mode selection switch #9 to off position after all calibration checks are complete or type Q and enter on keyboard.

SIGNATURE Jennifer A. Combs

*WITH OR WITHOUT VAPOR RECIRCULATION AND WITH OR WITHOUT KEYBOARD

DHS/DSLS/Form C116 (Rev. 7-93)

#15

CMI INC
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 64-003336
04/29/06

TEST	BAC VALUE	TIME
AIR BLANK	.000	15:29
INVALID SAMPLE	.XXX	15:26
AIR BLANK	.000	15:26

SUBJECT'S NAME

DULTO
INSTRUMENT LOCATION

TIME FIRST OBSERVED

46160

OPERATOR

ADDITIONAL INFORMATION AND/OR REMARKS

Sweet Breath



INTOXILYZER INSTRUMENT PRINTER CARD

© 1986 by CMI INC

#15

CMI INC
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 64-003336
04/29/06

TEST	BAC VALUE	TIME
AIR BLANK	.000	15:28
*SUBJECT TEST	.094	15:30
AIR BLANK	.000	15:30

* DEFICIENT SAMPLE - VALUE
PRINTED WAS HIGHEST OBTAINED.

SUBJECT'S NAME

DULTO
INSTRUMENT LOCATION

TIME FIRST OBSERVED

46160

OPERATOR

ADDITIONAL INFORMATION AND/OR REMARKS



INTOXILYZER INSTRUMENT PRINTER CARD

© 1986 by CMI INC

134 6 of 6 file



TUCSON POLICE DEPARTMENT
CRIME LABORATORY

1 of 1
file

INTOXILYZER SERVICE RECORD

BH

Date: April 20, 2006

QAS: Jennifer Coombs 46160

Intoxilyzer Serial Number: 64-003336 (#15)

Reason For Service: DVM reading is at the low end of the
DPS range of .200-.600.

Date Out of Service: 4/20/06

Calibration Check completed prior to removal from service

Date: 4/20/06

SQAP completed prior to removal from service

Date: 4/20/06

Corrective Action: Sent to Applied Electronics

