



CERTIFICATION by CUSTODIAN of RECORDS
Tucson Police Department Crime Laboratory

TB4

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 5000 Serial Number 66-001759 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R9-14-404.A.3, for a 0.080AC solution are 0.070AC to 0.090AC.

Date: 4/20/06 Cal 1 0.078 AC Cal 2 0.079 AC

Date: N/A Cal 1 AC Cal 2 AC

Date: N/A Cal 1 AC Cal 2 AC

Between the dates of N/A and N/A the instrument was operating properly and accurately.


The instrument was taken out of service for repair on 3/23/06.
 The instrument was returned to service on 4/20/06.

Jennifer L. Coombs
 Jennifer L. Coombs

5/2/06

ADPS QAS Permit # QAS 29886

SUBSCRIBED AND SWORN to before me this 2, day of May, 2006.

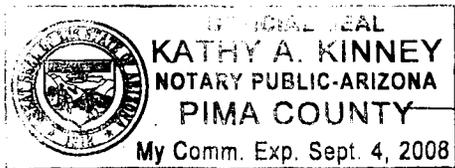

Kathy A. Kinney
 Notary Public

*Includes documents specified in A.A.C. R9-14-404.A.6

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.



MEMORANDUM

DATE: March 29, 2006

TO: Applied Electronics
52 Juniper Lane
P.O. Box 658
Eagle, CO 81631

FROM: **Jennifer L. Coombs**
Tucson Police Department
Crime Laboratory
270 S. Stone Avenue
Tucson, AZ 85701-1917

A handwritten signature in cursive script, appearing to read "JL Coombs".

SUBJECT: Intoxilyzer 5000 Maintenance and Repair

Enclosed is Intoxilyzer 66-001759.

Reason: The calibration checks have slightly increased.


Please repair.

CERTIFICATE OF CALIBRATION

*This is to certify that the calibration of INTOXILYZER®
serial number 66-001759, was tested and found to be in compliance
with the National Highway Traffic Safety Administration Standard
for Devices to Measure Breath Alcohol
(3.8 FR., No. 212 30459-30463 Nov. 5, 1973.)*

Date April 12, 2006

Signed [Signature]
Technician

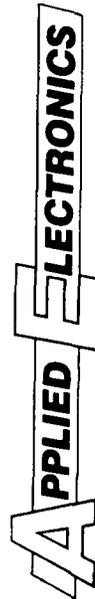

Factory Authorized Service Facility For CMI/MPH Products

Pursuant to section 28.1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.



• P.O. BOX 658 • EAGLE, CO 81631 Phone: (970) 328-5420

STATEMENT OF SERVICE WARRANTY

Applied Electronics warrants that any component having been replaced, to be free of defects in material and workmanship for a period of 90 days from date of delivery to its customer. This warranty is limited to only components that have been replaced during the repair of your product. Applied Electronics obligation is limited to repairing or replacing as we may elect of any component covered under this service warranty.

Any parts or components considered covered by this warranty shall be returned freight prepaid to Applied Electronics. Return freight shall be paid by Applied Electronics.

This warranty may be voided if the product has been adversely effected or tampered with in any way.

This service warranty is separate from the original manufacturers warranty. In no event shall Applied Electronics be liable for any loss or damages arising out of any such defective component or workmanship.

Pursuant to section 28.1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

EXHIBIT Q

1 of 6
jli

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 5000*

SH

AGENCY Tucson Police QA SPECIALIST Jennifer Combs 461160
(Print Name)

INTOXILYZER SERIAL # 166-001759 LOCATION LAB
DATE April 20 19 2006 TIME 0822

Display reads "READY TO START" or "PUSH BUTTON TO START TEST."

DIAGNOSTIC TESTS

DVM Test check. Setting should be between .200 and .600. Mode selection switch S2 on, S1 and S3 off or keyboard menu selection "H". Reading is .356.

Display Test check. Mode selection switch S1 on and S2 and S3 off or keyboard menu selection "V".

Printer Test check. Mode selection switch S1, S2, S3 off or keyboard menu selection "P".

Clock time check. Mode selection switch S10 on or keyboard menu selection "E".

Date check. Mode selection switch S11 on or keyboard menu selection "E".

OPERATIONAL TESTS

Alcohol free subject Test result 0.000 AC

Error recognition logic system functioning
Invalid test printed

Proper sample recognition system
Invalid sample printed ✓
Deficient sample printed ✓

N/A Completeness of sample purge with collection tube**

Calibration standard 0.080 AC

Results: 0.078 AC & 0.079 AC

Instrument operating properly and accurately. YES NO

COMMENTS:

SIGNATURE Jennifer J. Combs

0. N/A alcohol concentration standard collected for subsequent analysis.**

*WITH OR WITHOUT VAPOR RECIRCULATION AND WITH OR WITHOUT KEYBOARD
**THIS STEP IS ONLY REQUIRED IF THIS DEVICE IS BEING USED FOR
SAMPLE CAPTURE.

3 of 6
file

Tucson Police Department Crime Laboratory ^{TS4}

TUCSON POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-001759
04/20/86 08:49

TEST BRAC VALUE TIME
AIR BLANK .000 08:48
CAL. CHECK .079 08:49
AIR BLANK .000 08:49

SUBJECT'S NAME _____
LAB
INSTRUMENT LOCATION

TIME FIRST OBSERVED _____
46160
OPERATOR
ADDITIONAL INFORMATION AND/OR REMARKS



INTOXILYZER INSTRUMENT PRINTER CARD
© 1986 by CMI INC.

TUCSON POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-001759
04/20/86 08:48

TEST BRAC VALUE TIME
AIR BLANK .000 08:47
CAL. CHECK .079 08:47
AIR BLANK .000 08:48

SUBJECT'S NAME _____
LAB
INSTRUMENT LOCATION

TIME FIRST OBSERVED _____
46160
OPERATOR
ADDITIONAL INFORMATION AND/OR REMARKS



INTOXILYZER INSTRUMENT PRINTER CARD
© 1986 by CMI INC.

EXHIBIT P

5 of 6
file

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R9-14-404(A)

ARIZONA DEPARTMENT OF HEALTH SERVICES

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 5000*

STANDARD CALIBRATION CHECK PROCEDURE *B4*

Agency Tucson Police Department Date 4/20/06 Time 0822
Intoxilyzer Serial # 666-601759 Location LAB
QA Specialist Jennifer Coombs 461100
(Print Name)

- (✓) 1. Pour a standard alcohol solution of known value into a clean dry simulator jar and assemble the simulator. Insure that a tight seal has been made. Standard value: 0.050 AC
- (✓) 2. Turn on the simulator and allow the temperature to reach 34°C+.2°C.
- (✓) 3. Set Intoxilyzer mode selection in the ACA mode by switching mode selection switch #9 on or selecting "C" on keyboard menu.
- (✓) 4. Attach simulator to the simulator entrance port on the Intoxilyzer.
- ✓ (✓) 5. Intoxilyzer 5000 display reads "READY TO START" or "PUSH BUTTON".
- ✓ (✓) 6. Push Start Test button or press enter on keyboard.
- ✓ (✓) 7. Insert card in response to display.
- ✓ (✓) 8. Air blank completed.
- ✓ (✓) 9. Calibration check completed. Test results 0.078 AC # 0.079 AC
- ✓ (✓) 10. Air blank completed.
- ✓ (✓) 11. When display reads Test Complete remove evidence card. Attach the card to the completed checklist.
- (✓) 12. Return mode selection switch #9 to off position after all calibration checks are complete or type Q and enter on keyboard.

SIGNATURE Jennifer L Coombs

*WITH OR WITHOUT VAPOR RECIRCULATION AND WITH OR WITHOUT KEYBOARD

DHS/DSL/ Form C116 (Rev. 7-93)

TUCSON POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-001759
04/20/86 08:31

TEST BRAC VALUE TIME
AIR BLANK .000 08:28
*SUBJECT TEST .077 08:31
AIR BLANK .000 08:31
* DEFICIENT SAMPLE - VALUE
PRINTED WAS HIGHEST OBTAINED.

SUBJECT'S NAME

LAB
INSTRUMENT LOCATION

TIME FIRST OBSERVED

46160
OPERATOR

ADDITIONAL INFORMATION AND/OR REMARKS



INTOXILYZER INSTRUMENT PRINTER CARD

© 1986 by CMI INC

TUCSON POLICE DEPT.
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-001759
04/20/86 08:28

TEST BRAC VALUE TIME
AIR BLANK .000 08:27
INVALID SAMPLE .MOA 08:27
AIR BLANK .000 08:28
BEGIN NEW DEPRIVATION PERIOD

SUBJECT'S NAME

LAB
INSTRUMENT LOCATION

TIME FIRST OBSERVED

46160
OPERATOR

ADDITIONAL INFORMATION AND/OR REMARKS

Sweet Breath



INTOXILYZER INSTRUMENT PRINTER CARD

© 1986 by CMI INC

73
6 of 6
jki



TUCSON POLICE DEPARTMENT
CRIME LABORATORY

INTOXILYZER SERVICE RECORD

TS4

QAS: Jennifer Coombs 76160

Intoxilyzer Serial Number: 66-001759

Date Returned to Service: April 30, 2006