



CERTIFICATION by CUSTODIAN of RECORDS
Tucson Police Department Crime Laboratory

104

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 5000 Serial Number 166-004798 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.080AC solution are 0.070AC to 0.090AC.

Date: 9/1/06 Cal 1 0.080 AC Cal 2 0.082 AC

Date: 9/28/06 Cal 1 0.078 AC Cal 2 0.079 AC

Date: N/A Cal 1 _____ AC Cal 2 _____ AC

Between the dates of 9/1/06 and 9/28/06 the instrument was operating properly and accurately.


The instrument was taken out of service for repair on N/A.
The instrument was returned to service on N/A.

Jennifer L. Coombs
Jennifer L. Coombs

10/17/06

ADPS QAS Permit # QAS 29886

SUBSCRIBED AND SWORN to before me this 17, day of October, 2006.

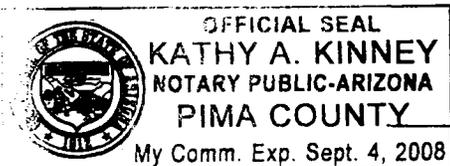

Kathy A. Kinney
Notary Public

*Includes documents specified in A.A.C. R13-10-104.A.5

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 5000*

1 of 6
file

SM

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Jennifer Coombs 46160 AGENCY Tucson Police Department
DATE September 28, 2006 TIME 0850
INTOXILYZER SERIAL # 166-004798 LOCATION ODD

(✓) 1. Display reads "READY TO START" or "PUSH BUTTON TO START TEST."

DIAGNOSTIC TESTS

- (✓) 1. DVM Test check. Setting should be between .010 and .600. Mode selection switch S2 on, S1 and S3 off or keyboard menu selection "H". Reading is .376.
- (✓) 2. Display Test check. Mode selection switch S1 on and S2 and S3 off or keyboard menu selection "V".
- (✓) 3. Printer Test check. Mode selection switch S1, S2, S3 off or keyboard menu selection "P".
- (✓) 4. Clock time check. Mode selection switch S10 on or keyboard menu selection "E".
- (✓) 5. Date check. Mode selection switch S11 on or keyboard menu selection "E".

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject Test result 0. 000 AC
- (✓) 2. Error recognition logic system functioning
Invalid test printed
- (✓) 3. Proper sample recognition system
Invalid sample printed ✓
Deficient sample printed ✓
- (✓) 4. Standard Calibration Check standard 0. 080 AC
Results: 0. 078 AC & 0. 079 AC

Instrument operating accurately and properly. YES NO

COMMENTS:

SIGNATURE Jennifer L. Coombs

*WITH OR WITHOUT VAPOR RECIRCULATION AND WITH OR WITHOUT KEYBOARD

DPS Form Exh E-4 (Rev. 05-1)

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

Tucson Police Department Crime Laboratory

3 of 6
file

TUCSON POLICE DEPARTMENT
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004798
09/28/06

TEST
AIR BLANK
CAL. CHECK
AIR BLANK

BRAC VALUE
.000
.079
.000

TIME
09:01
09:01
09:01

TIME FIRST OBSERVED _____
SUBJECT'S NAME _____
INSTRUMENT LOCATION **ODD**

46160
OPERATOR

ADDITIONAL INFORMATION AND/OR REMARKS



INTOXILYZER® INSTRUMENT PRINTER CARD
© 1986 by CMI INC.

TUCSON POLICE DEPARTMENT
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004798
09/28/06

TEST
AIR BLANK
CAL. CHECK
AIR BLANK

BRAC VALUE
.000
.078
.000

TIME
08:59
09:00
09:00

TIME FIRST OBSERVED _____
SUBJECT'S NAME _____
INSTRUMENT LOCATION **ODD**

46160
OPERATOR

ADDITIONAL INFORMATION AND/OR REMARKS



INTOXILYZER® INSTRUMENT PRINTER CARD
© 1986 by CMI INC.

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

By 4 of 6
jli

SN 66-004798
E767:14
INVALID TEST

09/28/06
08:54

SAMPLE INTRODUCED
AT IMPROPER TIME.

SUBJECT'S NAME

TIME FIRST OBSERVED

INSTRUMENT LOCATION

OPERATOR

ADDITIONAL INFORMATION AND/OR REMARKS

ODD

46160

INTOXILYZER[®] INSTRUMENT PRINTER CARD

© 1986 by CMI INC



EXHIBIT E-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

5 of 6
file

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 5000*

134

STANDARD CALIBRATION CHECK PROCEDURE

QA SPECIALIST Jennifer Coombs 46160 AGENCY Tucson Police Department
DATE September 28, 2006 TIME 0850
INTOXILYZER SERIAL # 66-004798 LOCATION ODD

- (✓) 1. Pour a standard alcohol concentration solution into a clean dry simulator jar and assemble the simulator. Ensure that a tight seal has been made. Standard value: 0. 080 AC.
- (✓) 2. Turn on the simulator and allow the temperature to reach 34°C ± 0.2°C.
- (✓) 3. Set Intoxilyzer mode selection in the ACA mode by switching mode selection switch #9 on or selecting "C" on keyboard menu.
- (✓) 4. Attach simulator to the simulator entrance port on the Intoxilyzer.
- ✓ ✓ 5. Intoxilyzer 5000 display reads "READY TO START" or "PUSH BUTTON".
- ✓ ✓ 6. Push Start Test button or press enter on keyboard.
- ✓ ✓ 7. Insert card in response to display.
- ✓ ✓ 8. Air blank completed.
- ✓ ✓ 9. Standard Calibration Check completed. Test results 0. 078 AC & 0. 079 AC.
- ✓ ✓ 10. Air blank completed.
- ✓ ✓ 11. When test complete, remove printed record. Attach the record to the completed checklist.
- (✓) 12. Return mode selection switch #9 to off position after all calibration checks are complete or type Q and enter on keyboard.

SIGNATURE Jennifer L. Coombs

*WITH OR WITHOUT VAPOR RECIRCULATION AND WITH OR WITHOUT KEYBOARD

DPS Form Exh E-2 (Rev. 05-1)

TUCSON POLICE DEPARTMENT
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004798
09/28/06 08:55

TEST BRAC VALUE TIME
AIR BLANK .000 08:54
INVALID SAMPLE .MDA 08:55
AIR BLANK .000 08:55

BEGIN NEW DEPRIVATION PERIOD

SUBJECT'S NAME

0DD

INSTRUMENT LOCATION

TIME FIRST OBSERVED

46160

OPERATOR

ADDITIONAL INFORMATION AND/OR REMARKS

Sweet Breath

INTOXILYZER® INSTRUMENT PRINTER CARD

© 1986 by CMI INC.

TUCSON POLICE DEPARTMENT
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004798
09/28/06 08:59

TEST BRAC VALUE TIME
AIR BLANK .000 08:56
*SUBJECT TEST .118 08:58
AIR BLANK .000 08:59

* DEFICIENT SAMPLE - VALUE
PRINTED WAS HIGHEST OBTAINED.

SUBJECT'S NAME

0DD

INSTRUMENT LOCATION

TIME FIRST OBSERVED

46160

OPERATOR

ADDITIONAL INFORMATION AND/OR REMARKS

INTOXILYZER® INSTRUMENT PRINTER CARD

© 1986 by CMI INC.

py 6 of 6 file

