



CERTIFICATION by CUSTODIAN of RECORDS
Tucson Police Department Crime Laboratory

[Handwritten signature]

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 5000 Serial Number 66-004922 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R9-14-404.A.3, for a 0.080AC solution are 0.070AC to 0.090AC.

Date: 5.11.06 Cal 1 0.077 AC Cal 2 0.077 AC

Date: 6.8.06 Cal 1 0.078 AC Cal 2 0.078 AC

Date: — Cal 1 — AC Cal 2 — AC

Between the dates of 5.11.06 and 6.8.06 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on —.
The instrument was returned to service on —.


[Handwritten signature: Terry S. Gallegos]

Terry S. Gallegos

6.13.2006

ADPS QAS Permit # QAS 976

SUBSCRIBED AND SWORN to before me this 13, day of June, 2006.

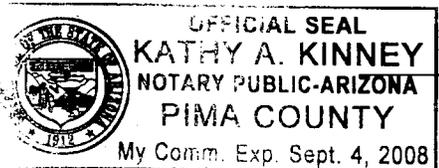

[Handwritten signature: Kathy A. Kinney]

Notary Public

*Includes documents specified in A.A.C. R9-14-404.A.6
Note: The applicable A.A.C. document sections were revised effective May 18, 2006 at 0900 hours. The new sections are A.A.C. R13-10-104.A.5 and A.A.C. R13-10-104.A.2

1066
TS4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 5000*

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Terry S. Gallegos 28947 AGENCY Tucson Police Department
DATE 06-08-2006 TIME 1446
INTOXILYZER SERIAL # 66-004922 LOCATION ODE

(✓) 1. Display reads "READY TO START" or "PUSH BUTTON TO START TEST."

DIAGNOSTIC TESTS

- (✓) 1. DVM Test check. Setting should be between .010 and .600. Mode selection switch S2 on, S1 and S3 off or keyboard menu selection "H". Reading is 0.380.
- (✓) 2. Display Test check. Mode selection switch S1 on and S2 and S3 off or keyboard menu selection "V".
- (✓) 3. Printer Test check. Mode selection switch S1, S2, S3 off or keyboard menu selection "P".
- (✓) 4. Clock time check. Mode selection switch S10 on or keyboard menu selection "E".
- (✓) 5. Date check. Mode selection switch S11 on or keyboard menu selection "E".

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject Test result 0.000 AC
- (✓) 2. Error recognition logic system functioning
Invalid test printed
- (✓) 3. Proper sample recognition system
Invalid sample printed ✓
Deficient sample printed ✓
- (✓) 4. Standard Calibration Check standard 0.080 AC
Results: 0.078 AC and 0.078 AC

Instrument operating accurately and properly. YES NO

COMMENTS: * 1501 → 1506

SIGNATURE Terry S Gallegos

*WITH OR WITHOUT VAPOR RECIRCULATION AND WITH OR WITHOUT KEYBOARD

Tucson Police Department Crime Laboratory

3056
154

TUCSON POLICE DEPARTMENT
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004922
06/08/06 14:51

TEST BRAC VALUE TIME
AIR BLANK .000 14:50
CAL. CHECK .078 14:49
AIR BLANK .000 14:49

TUCSON POLICE DEPARTMENT
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004922
06/08/06 14:50

TEST BRAC VALUE TIME
AIR BLANK .000 14:49
CAL. CHECK .078 14:49
AIR BLANK .000 14:49

SUBJECT'S NAME _____

ODE

INSTRUMENT LOCATION _____

TIME FIRST OBSERVED _____

Gallegos 28847

OPERATOR _____

ADDITIONAL INFORMATION AND/OR REMARKS _____



INTOXILYZER INSTRUMENT PRINTER CARD

© 1986 by CMI INC.

SUBJECT'S NAME _____

ODE

INSTRUMENT LOCATION _____

TIME FIRST OBSERVED _____

Gallegos 28847

OPERATOR _____

ADDITIONAL INFORMATION AND/OR REMARKS _____



INTOXILYZER INSTRUMENT PRINTER CARD

© 1986 by CMI INC.

4056
TS6

flu

SN 66-004922
E767.14
INVALID TEST

06/08/06
15:00

SAMPLE INTRODUCED
AT IMPROPER TIME.

SUBJECT'S NAME	
TIME FIRST OBSERVED	INSTRUMENT LOCATION
Gallegos	ODE
OPERATOR	23847
ADDITIONAL INFORMATION AND/OR REMARKS	

INTOXILYZER[®] INSTRUMENT PRINTER CARD

© 1986 by CMI INC.



THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

5066
DH

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 5000*

STANDARD CALIBRATION CHECK PROCEDURE *ju*

QA SPECIALIST Terry S. Gallegos 28847 AGENCY Tucson Police Department
DATE 06-08-2006 TIME 1446
INTOXILYZER SERIAL # 66-004922 LOCATION ODE

- (✓) 1. Pour a standard alcohol concentration solution into a clean dry simulator jar and assemble the simulator. Ensure that a tight seal has been made. Standard value: 0.080 AC.
- (✓) 2. Turn on the simulator and allow the temperature to reach 34°C ± 0.2°C.
- (✓) 3. Set Intoxilyzer mode selection in the ACA mode by switching mode selection switch #9 on or selecting "C" on keyboard menu.
- (✓) 4. Attach simulator to the simulator entrance port on the Intoxilyzer.
- (✓) 5. Intoxilyzer 5000 display reads "READY TO START" or "PUSH BUTTON".
- (✓) 6. Push Start Test button or press enter on keyboard.
- (✓) 7. Insert card in response to display.
- (✓) 8. Air blank completed.
- (✓) 9. Standard Calibration Check completed. Test results 0.078AC and 0.078 AC.
- (✓) 10. Air blank completed.
- (✓) 11. When test complete, remove printed record. Attach the record to the completed checklist.
- (✓) 12. Return mode selection switch #9 to off position after all calibration checks are complete or type Q and enter on keyboard.

SIGNATURE

Terry S. Gallegos

*WITH OR WITHOUT VAPOR RECIRCULATION AND WITH OR WITHOUT KEYBOARD

DPS Form Exh E-2 (Rev. 05-1)

TUCSON POLICE DEPARTMENT
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004922
06/08/06 14:55

TEST BRAC VALUE TIME
AIR BLANK .000 14:55
INVALID SAMPLE .MOA 14:55
AIR BLANK .000 14:55

BEGIN NEW DEPRIVATION PERIOD

SUBJECT'S NAME

ODE

TIME FIRST OBSERVED

INSTRUMENT LOCATION

Gallegos 28847

OPERATOR

ADDITIONAL INFORMATION AND/OR REMARKS

Sweet breath

INTOXILYZER INSTRUMENT PRINTER CARD

© 1986 by CMI INC



TUCSON POLICE DEPARTMENT
INTOXILYZER - ALCOHOL ANALYZER
AZ MODEL 5000 SN 66-004922
06/08/06 14:59

TEST BRAC VALUE TIME
AIR BLANK .000 14:59
*SUBJECT TEST .038 14:59
AIR BLANK .000 14:59

* DEFICIENT SAMPLE - VALUE
PRINTED WAS HIGHEST OBTAINED.

SUBJECT'S NAME

ODE

TIME FIRST OBSERVED

INSTRUMENT LOCATION

Gallegos 28847

OPERATOR

ADDITIONAL INFORMATION AND/OR REMARKS

INTOXILYZER INSTRUMENT PRINTER CARD

© 1986 by CMI INC



6076
Dh
ju