



CERTIFICATION by CUSTODIAN of RECORDS
Tucson Police Department Crime Laboratory

134
12-21-07

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-002803 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 12/14/07 Cal 1 0.100 AC Cal 2 100 AC

Between the dates of 11/21/07 and 12/14/07 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on -.
The instrument was returned to service on -.


Sarah L. Cione

Sarah L. Cione

12/19/07

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 19, day of December, 2007.

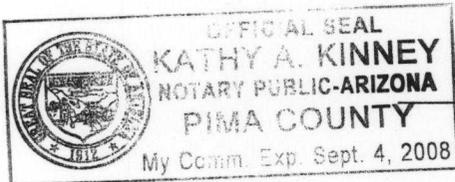

Kathy A Kinney

Notary Public

*Includes documents specified in A.A.C. R13-10-104.A.5

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

1 of 7 sc

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

BSH

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 12/14/07 TIME 10:46 am

INTOXILYZER SERIAL # 80-002803 LOCATION Team 2

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 100 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: ARIZONA
Serial Number : 80-002803
12/14/2007

10:48:24

2 of 75c
124

Standard Lot#: 725402I
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:49:14
Diagnostic Test	Pass	10:49:41
Air Blank	0.000	10:50:09
0.100 Cal Check	0.101	10:50:27
Air Blank	0.000	10:50:57
Subject Test	0.000	10:51:15
Air Blank	0.000	10:51:42
Five Minute Wait	ABT*	
Air Blank	0.000	10:52:18

*Sequence Aborted

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: ARIZONA
Serial Number : 80-002803
12/14/2007

10:52:27

3 of 7 sc

154

Standard Lot#: 725402I
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP,TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	10:53:00
Air Blank	IPS**	10:53:10

*RFI Detect

**Improper Sample

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: ARIZONA
Serial Number : 80-002803
12/14/2007

10:53:39

4 of 7sc

154

Standard Lot#: 725402I
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP,TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:54:51
Diagnostic Test	Pass	10:55:19
Air Blank	0.000	10:55:46
0.100 Cal Check	0.100	10:56:05
Air Blank	0.000	10:56:34
Subject Test	0.055*	10:59:43
Air Blank	0.000	11:00:14
Air Blank	0.000	11:00:42
Subject Test	XXX**	11:01:04
Air Blank	0.000	11:01:37

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: ARIZONA
Serial Number: 80-002803
12/14/2007 11:01:54


5 of 75C

124

Standard Lot#: 725402I
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	11:02:22
Cal Check	0.100	11:02:40
Air Blank	0.000	11:03:10

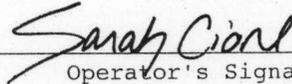
Operator's Signature

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6 of 7sc
154

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Sarah Cione AGENCY Tucson Police Department
DATE 12/14/07 TIME 10:46 am
Intoxilyzer Serial # 80-002803 Location Team 2

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C + 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: ARIZONA
Serial Number: 80-002803
12/14/2007 10:46:54

7 of 7sc
BY

Standard Lot#: 725402I
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	10:47:22
Cal Check	0.100	10:47:40
Air Blank	0.000	10:48:10

Sarah Cione
Operator's Signature



TUCSON POLICE DEPARTMENT
CRIME LABORATORY

1 of 2sc

INTOXILYZER SERVICE RECORD
FOR
CALIBRATION GAS TANK CHANGE

134

Date: 12/14/07

QAS: Sarah Cione

Intoxilyzer Serial Number: 80-002803

New Lot Number: 725402I

New Expiration Date: 9/12/2009

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."
Select "D" for dry gas and enter the following information:
Enter Target Value: 0.100
Enter Control Lot #: From the tank label
Enter Exp Date: From the tank label (mm/dd/yyyy)
Edit info Y/N? Y or N
Update Std Info? Y ENTER
"Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."
Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 100 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove and sign printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

INTOXILYZER 8000
Location: ARIZONA
Serial Number: 80-002803
12/14/2007 10:44:57

2 of 2 SC

SM

Standard Lot#: 725402I
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	10:45:24
Cal Check	0.100	10:45:43
Air Blank	0.000	10:46:13

Sarah Cione

Operator's Signature