

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

10/75C
TSY
12.13.07

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 11/21/07 TIME 9:14am

INTOXILYZER SERIAL # 80-000323 LOCATION Crime Lab - TPD

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 096 AC

Instrument is operating properly and accurately. YES NO

COMMENTS none

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-000323
11/21/2007 09:17:09

2 of 7 sc

154

Standard Lot#: 706601I
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:18:07
Diagnostic Test	Pass	09:18:34
Air Blank	0.000	09:19:02
0.100 Cal Check	0.098	09:19:21
Air Blank	0.000	09:19:50
Subject Test	0.000	09:20:14
Air Blank	0.000	09:20:41
Five Minute Wait	ABT*	
Air Blank	0.000	09:21:31

*Sequence Aborted

Not a Successfully Completed Test Sequence

Pursuant to section 16-227 Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-000323
11/21/2007

09:21:42

30 of 75c
sc
TS4

Standard Lot#: 706601I
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP,TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	09:22:33
Air Blank	IPS**	09:22:45

*RFI Detect
**Improper Sample

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-000323
11/21/2007

09:23:13

4 ~~0~~ of 7 SC
SC

TS4

Standard Lot#: 706601I
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	09:24:03
Diagnostic Test	Pass	09:24:30
Air Blank	0.000	09:24:57
0.100 Cal Check	0.098	09:25:16
Air Blank	0.000	09:25:46
Subject Test	0.212*	09:28:54
Air Blank	0.000	09:29:28
Air Blank	0.000	09:29:55
Subject Test	XXX**	09:30:24
Air Blank	0.000	09:30:58

*Deficient Sample

**Invalid Sample

Begin new deprivation period

INTOXILYZER 8000

Location: TUCSON POLICE DEPT

Serial Number: 80-000323

11/21/2007

09:31:56

Standard Lot#: 706601I

Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616

TUCSON PD

5 of 7 SC
SC
DH

Test	g/210L	Time
Air Blank	0.000	09:32:23
Cal Check	0.096	09:32:42
Air Blank	0.000	09:33:11

Sarah Cione

Operator's Signature

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD CALIBRATION CHECK PROCEDURE

6 of 7 sc

PH

QA Specialist Sarah Cione AGENCY Tucson Police Department
DATE 11/21/07 TIME 9:14 am
Intoxilyzer Serial # 80-000323 Location Crime Lab - TPD

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 098 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000

Location: TUCSON POLICE DEPT

Serial Number: 80-000323

11/21/2007

09:14:53

7 of 750

Standard Lot#: 706601I

Last Changed By: S. CIONE #50616

134

QAS: S. CIONE #50616

TUCSON PD

Test	g/210L	Time
Air Blank	0.000	09:15:20
Cal Check	0.098	09:15:39
Air Blank	0.000	09:16:08

Sarah Cione

Operator's Signature