



CERTIFICATION by CUSTODIAN of RECORDS

Tucson Police Department Crime Laboratory
270 S. Stone Avenue Tucson, AZ 85701-1917

BH
2.4.2009

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-000323 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 1/9/09 Cal 1 0.098 AC Cal 2 0.097 AC

Between the dates of 12/12/08 and 1/9/09 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on _____.
The instrument was returned to service on _____.

Sarah Cione

Sarah L. Cione


1/12/09

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 12th, day of January, 2009.

A. Hernandez

Notary Public

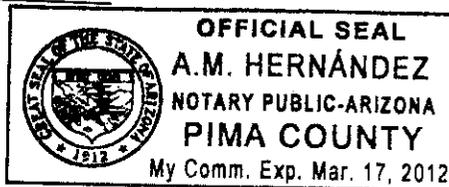

*Includes documents specified in A.A.C. R13-10-104.A.5

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

1 of 7 sc

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

By
2.4.2009

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 1/9/09 TIME 13:15

INTOXILYZER SERIAL # 80-000323 LOCATION Crime Lab

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 097 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

2 of 7sc

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-000323
01/09/2009

13:17:28

TS4

Standard Lot#: 719802I
Last Changed By: J. TRAINING #99999

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:18:15
Diagnostic Test	Pass	13:18:42
Air Blank	0.000	13:19:10
0.100 Cal Check	0.099	13:19:28
Air Blank	0.000	13:19:58
Subject Test	0.000	13:20:28
Air Blank	0.000	13:20:56
Five Minute Wait	ABT*	
Air Blank	0.000	13:21:29

*Sequence Aborted

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

3 of 7 sc

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-000323
01/09/2009 13:21:34

B4

Standard Lot#: 719802I
Last Changed By: J. TRAINING #99999

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SOAP,TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	13:22:11
Air Blank	IPS**	13:22:20

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-000323
01/09/2009

13:22:34

4 of 7sc

134

Standard Lot#: 719802I
Last Changed By: J. TRAINING #99999

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:23:26
Diagnostic Test	Pass	13:23:53
Air Blank	0.000	13:24:21
0.100 Cal Check	0.098	13:24:39
Air Blank	0.000	13:25:09
Subject Test	0.129*	13:28:17
Air Blank	0.000	13:28:49
Air Blank	0.000	13:29:16
Subject Test	XXX**	13:29:37
Air Blank	0.000	13:30:10

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-000323
01/09/2009 13:30:28

5 of 7 SC

TS4

Standard Lot#: 719802I
Last Changed By: J. TRAINING #99999

QAS: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	13:30:55
Cal Check	0.097	13:31:14
Air Blank	0.000	13:31:44

Sarah Cione

Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6 of 7 SC

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

134
2.4.09

QA Specialist Sarah Cione AGENCY Tucson Police Department
DATE 1/9/09 TIME 13:15
Intoxilyzer Serial # 80-000323 Location Crime Lab

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 098 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-000323
01/09/2009 13:15:49

7 of 7 sc

BH

Standard Lot#: 719802I
Last Changed By: J. TRAINING #99999

QAS: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	13:16:16
Cal Check	0.098	13:16:35
Air Blank	0.000	13:17:04

Sarah Cione
Operator's Signature