



CERTIFICATION by CUSTODIAN of RECORDS  
Tucson Police Department Crime Laboratory

I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 8000 Serial Number 80-000332 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

Tsh

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 1/19/07 Cal 1 0.100 AC Cal 2 0.100 AC

Between the dates of 12/22/06 and 1/19/07 the instrument was operating properly and accurately.


The instrument was taken out of service for repair on N/A.  
The instrument was returned to service on N/A.

Jennifer L. Coombs  
Jennifer L. Coombs

2/9/07

ADPS QAS Permit # 33120

SUBSCRIBED AND SWORN to before me this 9, day of February, 2007.

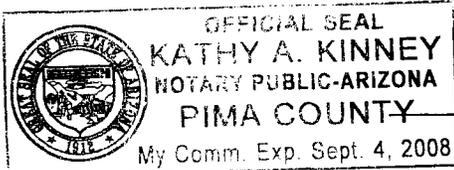

Kathy A. Kinney  
Notary Public

\*Includes documents specified in A.A.C. R13-10-104.A.5

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

1 of 9  
file  
By

QA SPECIALIST Jennifer Combs 46160 AGENCY Tucson Police Department

DATE January 19, 2007 TIME 1331

INTOXILYZER SERIAL # 80-000332 LOCATION ODE

( ✓ ) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- ( ✓ ) 1. Clock time check.
- ( ✓ ) 2. Date check.

OPERATIONAL TESTS

- ( ✓ ) 1. Alcohol-free subject test result 0. 000 AC.
- ( ✓ ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed ✓
- ( ✓ ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed ✓  
Deficient sample printed.
- ( ✓ ) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 100 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS Gas tank change prior to G-2.  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE Jennifer L. Combs

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-000332  
01/19/2007

13:38:56

2 of 9  
me  
TS4

Standard Lot#: 630601I  
Last Changed By: J. COOMBS #46160

QAS: J. COOMBS #46160  
TUCSON PD CRIME LAB

Operator: J. COOMBS #46160  
TUCSON PD CRIME LAB

Subject: SQAP,TEST  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:40:41
Diagnostic Test	Pass	13:41:08
Air Blank	0.000	13:41:36
0.100 Cal Check	0.101	13:41:54
Air Blank	0.000	13:42:24
Subject Test	0.000	13:42:49
Air Blank	0.000	13:43:16
Five Minute Wait	ABT*	
Air Blank	0.000	13:44:06

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-000332  
01/19/2007

13:44:36

3 of 9  
TS4

---

Standard Lot#: 630601I  
Last Changed By: J. COOMBS #46160

---

QAS: J. COOMBS #46160  
TUCSON PD CRIME LAB

---

Operator: J. COOMBS #46160  
TUCSON PD CRIME LAB

---

Subject: SQAP,TEST  
DOB: 11/11/1111  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	IPS*	13:45:33
Air Blank	0.000	13:46:02

\*Improper Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-000332  
01/19/2007

13:46:22

4 of 9  
file  
B4

Standard Lot#: 630601I  
Last Changed By: J. COOMBS #46160

QAS: J. COOMBS #46160  
TUCSON PD CRIME LAB

Operator: J. COOMBS #46160  
TUCSON PD CRIME LAB

Subject: SQAP,TEST  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:47:22
Diagnostic Test	Pass	13:47:49
Air Blank	0.000	13:48:17
0.100 Cal Check	RFI*	13:48:34
Air Blank	0.000	13:49:05

\*RFI Detect

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-000332  
01/19/2007

14:16:51

5 of 9  
file  
by

---

Standard Lot#: 630601I  
Last Changed By: J. COOMBS #46160

---

QAS: J. COOMBS #46160  
TUCSON PD CRIME LAB

---

Operator: J. COOMBS #46160  
TUCSON PD CRIME LAB

---

Subject: SQAP, TEST  
DOB: 11/11/1111  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	14:17:55
Diagnostic Test	Pass	14:18:22
Air Blank	0.000	14:18:49
0.100 Cal Check	0.100	14:19:08
Air Blank	0.000	14:19:38
Subject Test	XXX*	14:19:56
Air Blank	0.000	14:20:29

\*Invalid Sample  
Begin new deprivation period

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-000332  
01/19/2007

13:49:11

leaf 9  
file  
154

---

Standard Lot#: 630601I  
Last Changed By: J. COOMBS #46160

---

QAS: J. COOMBS #46160  
TUCSON PD CRIME LAB

---

Operator: J. COOMBS #46160  
TUCSON PD CRIME LAB

---

Subject: SQAP,TEST  
DOB: 11/11/1111  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	13:50:11
Diagnostic Test	Pass	13:50:38
Air Blank	0.000	13:51:06
0.100 Cal Check	0.100	13:51:24
Air Blank	0.000	13:51:54
Subject Test	0.020*	13:55:03
Air Blank	0.000	13:55:32
Air Blank	0.000	13:56:00
Subject Test	0.000	13:57:27
Air Blank	0.000	13:57:54
Five Minute Wait		
Air Blank	0.000	14:02:55
Subject Test	DEF**	14:06:03
Air Blank	0.000	14:06:31

\*Deficient Sample  
\*\*Deficient Sample

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-000332  
01/19/2007 14:22:25

7 of 9  
TS4  
JL

-----  
Standard Lot#: 630601I  
Last Changed By: J. COOMBS #46160  
-----

QAS: J. COOMBS #46160  
TUCSON PD CRIME LAB  
-----

Test	g/210L	Time
Air Blank	0.000	14:22:53
Cal Check	0.100	14:23:12
Air Blank	0.000	14:23:41

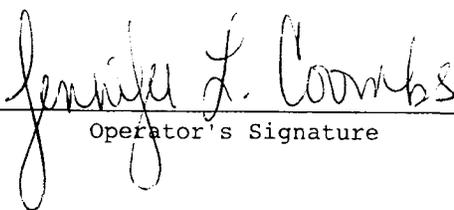
  
-----  
Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

8 of 9  
file  
B4

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Jennifer Combs 46160 AGENCY Tucson Police Department  
DATE January 19, 2007 TIME 1331  
Intoxilyzer Serial # 80-000332 Location ODE

- ( ✓ ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- ( ✓ ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ( ✓ ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ( ✓ ) 4. Air blank completed.
- ( ✓ ) 5. Calibration check completed. Test results 0. 100 AC.
- ( ✓ ) 6. Air blank completed.
- ( ✓ ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Jennifer L. Combs

DPS Form Exh G-2 (Rev. 05-01)

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.


INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-000332  
01/19/2007 13:31:25

989  
By JLC

-----  
Standard Lot#: 630601I  
Last Changed By: J. COOMBS #46160  
-----

QAS: J. COOMBS #46160  
TUCSON PD CRIME LAB  
-----

Test	g/210L	Time
Air Blank	0.000	13:31:52
Cal Check	0.100	13:32:11
Air Blank	0.000	13:32:41

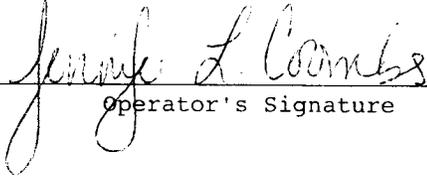  
\_\_\_\_\_  
Operator's Signature



TUCSON POLICE DEPARTMENT  
CRIME LABORATORY

1 of 1  
file

INTOXILYZER SERVICE RECORD  
FOR  
CALIBRATION GAS TANK CHANGE

104

Date: 1/19/07  
QAS: Jennifer Coombs 46160  
Intoxilyzer Serial Number: 80-000332  
New Lot Number: 630601I  
New Expiration Date: 11/06/2008

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."  
Select "D" for dry gas and enter the following information:  
Enter Target Value: 0.100  
Enter Control Lot #: From the tank label  
Enter Exp Date: From the tank label (mm/dd/yyyy)  
Edit info Y/N? Y or N  
Update Std Info? Y ENTER  
"Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."  
Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 100 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove and sign printed record. Attach the record to the completed checklist.

SIGNATURE Jennifer L. Coombs