



CERTIFICATION by CUSTODIAN of RECORDS  
Tucson Police Department Crime Laboratory

BS4 12-13-07

I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 8000 Serial Number 80-000712 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 11/19/07 Cal 1 0.100 AC Cal 2 0.099 AC

Between the dates of 10/19/07 and 11/19/07 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on N/A.  
The instrument was returned to service on N/A.


Clare Conley

Clare L. Conley

12-7-07

ADPS QAS Permit # 33314

SUBSCRIBED AND SWORN to before me this 7, day of December, 2007.

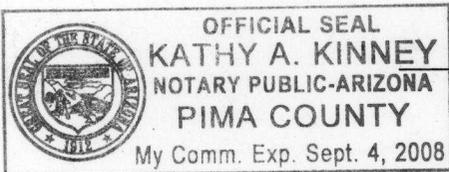

Kathy A Kinney  
Notary Public

\*Includes documents specified in A.A.C. R13-10-104.A.5

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

1 of 7 cu

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

B6  
12-13-07

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Clare Conley AGENCY Tucson Police Department  
DATE 11-19-07 TIME 10:39  
INTOXILYZER SERIAL # 80-800712 LOCATION DUI Headquarters

( ✓ ) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- ( ✓ ) 1. Clock time check.
- ( ✓ ) 2. Date check.

OPERATIONAL TESTS

- ( ✓ ) 1. Alcohol-free subject test result 0. 000 AC.
- ( ✓ ) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- ( ✓ ) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- ( ✓ ) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 099 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS None.  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE Clare Conley

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-000712  
11/19/2007 10:41:58

2 of 7 cu  
B4

---

Standard Lot#: 719802I  
Last Changed By: S. CLEARY #44524

---

QAS: C. CONLEY #34517  
TUCSON PD

---

Operator: C. CONLEY #34517  
TUCSON PD

---

Subject: TEST, SQAP  
DOB: 11/11/1111  
Sex: M Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	10:42:48
Diagnostic Test	Pass	10:43:15
Air Blank	0.000	10:43:43
0.100 Cal Check	0.100	10:44:01
Air Blank	0.000	10:44:31
Subject Test	0.000	10:44:47
Air Blank	ABT*	10:44:53
Air Blank	0.000	10:45:22

\*Sequence Aborted

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-000712  
11/19/2007

10:45:26

3 of 7 cu  
134

---

Standard Lot#: 719802I  
Last Changed By: S. CLEARY #44524

---

QAS: C. CONLEY #34517  
TUCSON PD

---

Operator: C. CONLEY #34517  
TUCSON PD

---

Subject: TEST, SQAP  
DOB: 11/11/1111  
Sex: M Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	RFI*	10:45:57
Air Blank	IPS**	10:46:11

\*RFI Detect  
\*\*Improper Sample

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number : 80-000712  
11/19/2007

10:46:20

4 of 7 cu  
TS4

Standard Lot#: 719802I  
Last Changed By: S. CLEARY #44524

QAS: C. CONLEY #34517  
TUCSON PD

Operator: C. CONLEY #34517  
TUCSON PD

Subject: TEST, SQAP  
DOB: 11/11/1111  
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	10:47:13
Diagnostic Test	Pass	10:47:40
Air Blank	0.000	10:48:08
0.100 Cal Check	0.101	10:48:26
Air Blank	0.000	10:48:56
Subject Test	0.225*	10:52:04
Air Blank	0.000	10:52:37
Air Blank	0.000	10:53:04
Subject Test	XXX**	10:53:49
Air Blank	0.000	10:54:22

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000

Location: TUCSON POLICE DEPT

Serial Number: 80-000712

11/19/2007

10:55:11

556700

T24

-----  
Standard Lot#: 719802I

Last Changed By: S. CLEARY #44524  
-----

QAS: C. CONLEY #34517

TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	10:55:38
Cal Check	0.099	10:55:57
Air Blank	0.000	10:56:26

*Clare Conley*

Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

By Bof7an

QA Specialist Clare Conley AGENCY Tucson Police Department  
DATE 11-19-07 TIME 10:39  
Intoxilyzer Serial # 80-000712 Location DUI Headquarters

- ( ✓ ) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- ( ✓ ) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- ( ✓ ) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- ( ✓ ) 4. Air blank completed.
- ( ✓ ) 5. Calibration check completed. Test results 0. 100 AC.
- ( ✓ ) 6. Air blank completed.
- ( ✓ ) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Clare Conley

DPS Form Exh G-2 (Rev. 05-01)

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-000712  
11/19/2007

10:38:46

7 of 7 cu

154

-----  
Standard Lot#: 719802I  
Last Changed By: S. CLEARY #44524  
-----

QAS: C. CONLEY #34517  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	10:39:13
Cal Check	0.100	10:39:32
Air Blank	0.000	10:40:01

*Clare Conley*

Operator's Signature



TUCSON POLICE DEPARTMENT  
CRIME LABORATORY

INTOXILYZER SERVICE RECORD  
FOR  
CALIBRATION GAS TANK CHANGE

By 12-13-07

1 of 2 154

Date: 11/03/07  
 QAS: CLARY 44524  
 Intoxilyzer Serial Number: 80-000712  
 New Lot Number: 719802I  
 New Expiration Date: 07/23/09

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."  
 Select "D" for dry gas and enter the following information:  
     Enter Target Value: 0.100  
     Enter Control Lot #: From the tank label  
     Enter Exp Date: From the tank label (mm/dd/yyyy)  
     Edit info Y/N? Y or N  
     Update Std Info? Y ENTER  
     "Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."  
 Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 100 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove and sign printed record. Attach the record to the completed checklist.

SIGNATURE Sam Clary 44524

# Tucson Police Department Crime Laboratory

2 of 2  
TS  
B4

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-000712  
11/03/2007 18:41:26

-----  
Standard Lot#: 7198021  
Last Changed By: S. CLEARY #44524  
-----

OAS: S. CLEARY #44524  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	18:41:53
Cal Check	0.100	18:42:12
Air Blank	0.000	18:42:41

  
-----  
Operator's Signature