



CERTIFICATION by CUSTODIAN of RECORDS
 Tucson Police Department Crime Laboratory
 270 S. Stone Avenue Tucson, AZ 85701-1917

TS4
 2-4-2009

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-001165 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 1/9/09 Cal 1 0.099 AC Cal 2 0.099 AC

Between the dates of 12/2/08 and 1/9/09 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on —.
 The instrument was returned to service on —.

Sarah Cione

Sarah L. Cione


1/12/09

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 12th day of January, 2009.

A. M. Hernandez

Notary Public

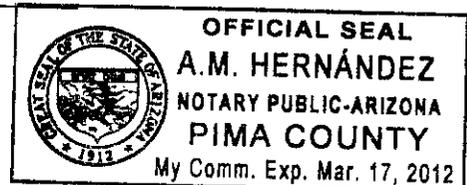

*Includes documents specified in A.A.C. R13-10-104.A.5

1 of 7 SC

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

TS4
2.4.2009

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department
DATE 1/9/09 TIME 10:59
INTOXILYZER SERIAL # 80-001165 LOCATION Team 1-ODS

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 099 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE

Sarah Cione

2 of 7sc

154

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-001165
01/09/2009 11:00:33

Standard Lot#: 804302I
Last Changed By: S. -CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:01:15
Diagnostic Test	Pass	11:01:43
Air Blank	0.000	11:02:10
0.100 Cal Check	0.099	11:02:29
Air Blank	0.000	11:02:58
Subject Test	0.000	11:03:16
Air Blank	0.000	11:03:44
Five Minute Wait	ABT*	
Air Blank	0.000	11:04:18

*Sequence Aborted

Not a Successfully Completed Test Sequence

3 of 7 sc

BY

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-001165
01/09/2009 11:04:26

Standard Lot#: 804302I
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	11:04:56
Air Blank	IPS**	11:05:05

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

4 of 7_{sc}
154

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-001165
01/09/2009 11:05:10

Standard Lot#: 804302I
Last Changed By: S. -CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	11:05:56
Diagnostic Test	Pass	11:06:24
Air Blank	0.000	11:06:51
0.100 Cal Check	0.099	11:07:10
Air Blank	0.000	11:07:39
Subject Test	0.058*	11:10:48
Air Blank	0.000	11:11:20
Air Blank	0.000	11:11:47
Subject Test	XXX**	11:12:01
Air Blank	0.000	11:12:34

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-001165
01/09/2009 11:13:06

5 of 7sc

TS4

Standard Lot#: 804302I
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	11:13:33
Cal Check	0.099	11:13:52
Air Blank	0.000	11:14:23

Sarah Cione
Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6 of 7 SC

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

TSY
2.4.2009

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Sarah Cione AGENCY Tucson Police Department
DATE 1/9/09 TIME 10:59
Intoxilyzer Serial # 80-001165 Location Team 1- ODS

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-001165
01/09/2009 10:59:07


7 of 7 SC

TS4

Standard Lot#: 804302I
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	10:59:34
Cal Check	0.099	10:59:53
Air Blank	0.000	11:00:22

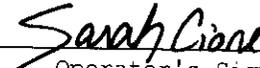

Operator's Signature