



CERTIFICATION by CUSTODIAN of RECORDS
 Tucson Police Department Crime Laboratory
 270 S. Stone Avenue Tucson, AZ 85701-1917

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-001165 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

154
317.01

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 3/6/09 Cal 1 0.099 AC Cal 2 0.099 AC

Between the dates of 2/6/09 and 3/6/09 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on -.
 The instrument was returned to service on -.

Sarah Cione

Sarah L. Cione


3/17/09

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 17, day of March, 20 09.

A.M. Hernandez

Notary Public

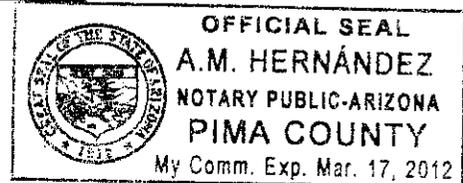

*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

1 of 7 sc

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD QUALITY ASSURANCE PROCEDURE

154
3-17-09

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department
DATE 3/6/09 TIME 13:53
INTOXILYZER SERIAL # 80-001165 LOCATION Team 1

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 099 AC

Instrument is operating properly and accurately. YES NO

COMMENTS

SIGNATURE Sarah Cione

2 of 7sc

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-001165
03/06/2009

13:55:03

154

Standard Lot#: 804302I
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP,TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:55:46
Diagnostic Test	Pass	13:56:14
Air Blank	0.000	13:56:41
0.100 Cal Check	0.099	13:57:00
Air Blank	0.000	13:57:32
Subject Test	0.000	13:57:48
Air Blank	0.000	13:58:15
Five Minute Wait	ABT*	
Air Blank	0.000	13:58:59

*Sequence Aborted

Not a Successfully Completed Test Sequence

3 of 7_{sc}

154

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-001165
03/06/2009 13:59:04

Standard Lot#: 804302I
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP,TEST
DOB: 11/11/1111
Sex: F Weight: .111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	13:59:37
Air Blank	IPS**	13:59:45

*RFI Detect
**Improper Sample

Not a Successfully Completed Test Sequence

4 of 7 SC

154

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-001165
03/06/2009 13:59:51

Standard Lot#: 804302I
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616
TUCSON PD

Operator: S. CIONE #50616
TUCSON PD

Subject: SQAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	14:00:33
Diagnostic Test	Pass	14:01:01
Air Blank	0.000	14:01:28
0.100 Cal Check	0.098	14:01:47
Air Blank	0.000	14:02:17
Subject Test	0.119*	14:05:25
Air Blank	0.000	14:05:58
Air Blank	0.000	14:06:25
Subject Test	XXX**	14:06:53
Air Blank	0.000	14:07:26

*Deficient Sample
**Invalid Sample
Begin new deprivation period

Not a Successfully Completed Test Sequence

5 of 7 SC

154

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-001165
03/06/2009 14:07:43

Standard Lot#: 804302I
Last Changed By: S. CIONE #50616

QAS:- S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	14:08:11
Cal Check	0.099	14:08:29
Air Blank	0.000	14:08:59

Sarah Cione

Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

6 of 7_{5c}

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

TSY
3-17-09

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Sarah Cione AGENCY Tucson Police Department
DATE 3/6/09 TIME 13:53
Intoxilyzer Serial # 80-001165 Location Team 1

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 099 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE

Sarah Cione

DPS Form Exh G-2 (Rev. 05-01)

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number: 80-001165
03/06/2009 13:53:41

7 of 7 sc

Standard Lot#: 804302I
Last Changed By: S. CIONE #50616

154

QAS: S. CIONE #50616
TUCSON PD

Test	g/210L	Time
Air Blank	0.000	13:54:09
Cal Check	0.099	13:54:28
Air Blank	0.000	13:54:57

Sarah Cione

Operator's Signature