



CERTIFICATION by CUSTODIAN of RECORDS
Tucson Police Department Crime Laboratory

[Handwritten signature]

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-001199 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 12-1-2006 Cal 1 0.097 AC Cal 2 0.097 AC


The instrument was placed into service on 12-1-2006.

Terry Gallegos
Terry S. Gallegos

1.4.2007

ADPS QAS Permit # 33169

SUBSCRIBED AND SWORN to before me this 4, day of January, 20 07.

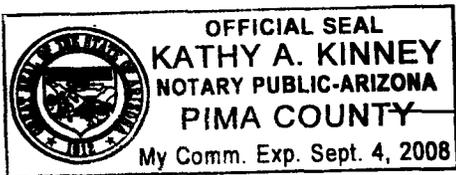

Kathy A Kinney

Notary Public

*Includes documents specified in A.A.C. R13-10-104.A.5

1 of 9
TS4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

je

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Terry Gallegos 20047 AGENCY Tucson Police Department

DATE 12-1-2006 TIME 1514

INTOXILYZER SERIAL # 80-001199 LOCATION Lab

() 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- () 1. Clock time check.
- () 2. Date check.

OPERATIONAL TESTS

- () 1. Alcohol-free subject test result 0.000 AC.
- () 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- () 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- () 4. Standard Calibration Check standard 0.110 AC
Result: 0.097 AC

Instrument is operating properly and accurately. YES NO

COMMENTS Diagnostics fail due to RFI during test.

SIGNATURE *Terry Gallegos*

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

2 of 9
TS4

INTOXILYZER 8000
Location: ARIZONA
Serial Number : 80-001199
12/01/2006

15:15:38

Standard Lot#: 502602I
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Operator: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Subject: SQAP,TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:16:32
Diagnostic Test	Pass	15:17:00
Air Blank	0.000	15:17:27
0.100 Cal Check	0.097	15:17:46
Air Blank	0.000	15:18:15
Subject Test	0.000	15:19:20
Air Blank	0.000	15:19:48
Five Minute Wait	ABT*	
Air Blank	0.000	15:20:56

*Sequence Aborted

Not a Successfully Completed Test Sequence

30f9
T54

INTOXILYZER 8000
Location: ARIZONA
Serial Number : 80-001199
12/01/2006

15:21:15

Standard Lot#: 502602I
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Operator: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Subject: SQAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:23:06
Diagnostic Test	Fail*	15:23:09
Air Blank	IPS**	15:23:21

* Voltage/Current Test
**Improper Sample

Not a Successfully Completed Test Sequence

4 of 9
736

INTOXILYZER 8000
Location: ARIZONA
Serial Number : 80-001199
12/01/2006

15:23:51

Standard Lot#: 502602I
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Operator: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Subject: SQAP,TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:24:47
Diagnostic Test	Fail*	15:24:50
Air Blank	0.000	15:25:18

* Analytical Stability Test

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

5089
TG

INTOXILYZER 8000
Location: ARIZONA
Serial Number : 80-001199
12/01/2006 15:25:49

Standard Lot#: 502602I
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Operator: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Subject: SQAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:26:45
Diagnostic Test	Pass	15:27:13
Air Blank	0.000	15:27:40
0.100 Cal Check	RFI*	15:27:56
Air Blank	0.000	15:28:27

*RFI Detect

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

6089
TSL

INTOXILYZER 8000
Location: ARIZONA
Serial Number : 80-001199
12/01/2006 15:28:41

Standard Lot#: 502602I
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Operator: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Subject: SQAP, TEST
DOB: 11/11/1111
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	15:29:34
Diagnostic Test	Pass	15:30:02
Air Blank	0.000	15:30:29
0.100 Cal Check	0.097	15:30:48
Air Blank	0.000	15:31:17
Subject Test	0.169*	15:34:25
Air Blank	0.000	15:34:57
Air Blank	0.000	15:35:24
Subject Test	XXX**	15:35:45
Air Blank	0.000	15:36:18

*Deficient Sample
**Invalid Sample
Begin new deprivation period

Not a Successfully Completed Test Sequence

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

7 of 9
TS4

INTOXILYZER 8000
Location: ARIZONA
Serial Number: 80-001199
12/01/2006

15:39:58

Standard Lot#: 502602I
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Test	g/210L	Time
Air Blank	0.000	15:40:25
Cal Check	0.097	15:40:44
Air Blank	0.000	15:41:14



Operator's Signature

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE *je*

QA Specialist Terry Gallegos 28847 AGENCY Tucson Police Department
DATE 12.1.2006 TIME 1514
Intoxilyzer Serial # 80-001199 Location Lab

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach $34^{\circ}\text{C} \pm 0.2^{\circ}\text{C}$
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 097 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE *Terry Gallegos*

DPS Form Exh G-2 (Rev. 05-01)


9 of 9
TS4

INTOXILYZER 8000
Location: ARIZONA
Serial Number: 80-001199
12/01/2006 15:13:52

Standard Lot#: 502602I
Last Changed By: T. GALLEGOS #28847

QAS: T. GALLEGOS #28847
TUCSON PD CRIME LAB

Test	g/210L	Time
Air Blank	0.000	15:14:19
Cal Check	0.097	15:14:38
Air Blank	0.000	15:15:07

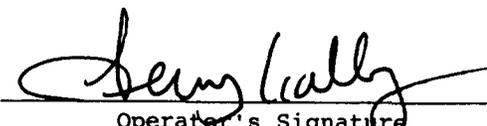

Operator's Signature



TUCSON POLICE DEPARTMENT
CRIME LABORATORY

jk

INTOXILYZER SERVICE RECORD

10f1
TS4

Intoxilyzer Serial Number: 80-001199

Initial Service Date: 12.1.2006

CMI, Inc. Certificate of Calibration Date: 7.23.2005