



CERTIFICATION by CUSTODIAN of RECORDS
Tucson Police Department Crime Laboratory

TS4
9.25.08

I hereby certify that these documents are a true and correct copy of the periodic maintenance* for the Intoxilyzer 8000 Serial Number 80-001199 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 9-19-08 Cal 1 0.101 AC Cal 2 0.098 AC

Between the dates of 8-27-08 and 9-19-08 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on N/A.
The instrument was returned to service on N/A.

Clare Conley

Clare L. Conley


9-25-08

ADPS QAS Permit # 33314

SUBSCRIBED AND SWORN to before me this 25th, day of September, 2008.

A.M. Hernandez

Notary Public

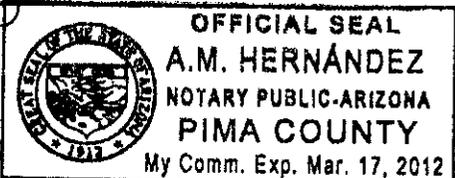

*Includes documents specified in A.A.C. R13-10-104.A.5

EXHIBIT G-4

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY
STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000
STANDARD QUALITY ASSURANCE PROCEDURE

1567
en
BY 9-25-08

QA SPECIALIST Clare Bonley AGENCY Tucson Police Department
DATE 9-19-08 TIME 0704
INTOXILYZER SERIAL # 80-001199 LOCATION Crime Lab

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (-) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.
Not a Successfully Completed Test Sequence printed
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC
Result: 0. 098 AC

Instrument is operating properly and accurately. YES NO

COMMENTS None

SIGNATURE Clare Bonley

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-001199
09/19/2008

07:12:54

2587
Cw
Tsh

Standard Lot#: 801002I
Last Changed By: S. CIONE #50616

QAS: C. CONLEY #34517
TUCSON PD

Operator: C. CONLEY #34517
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1111
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:13:49
Diagnostic Test	Pass	07:14:16
Air Blank	0.000	07:14:44
0.100 Cal Check	0.099	07:15:02
Air Blank	0.000	07:15:32
Subject Test	0.000	07:15:57
Air Blank	ABT*	07:16:03
Air Blank	0.000	07:16:32

*Sequence Aborted

Not a Successfully Completed Test Sequence

INTOXILYZER 8000
Location: TUCSON POLICE DEPT
Serial Number : 80-001199
09/19/2008

07:16:36

Standard Lot#: 801002I
Last Changed By: S. CIONE #50616

QAS: C. CONLEY #34517
TUCSON PD

Operator: C. CONLEY #34517
TUCSON PD

Subject: TEST, SQAP
DOB: 11/11/1111
Sex: M Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	RFI*	07:17:08
Air Blank	IPS**	07:17:15

*RFI Detect
**Improper Sample

347
en
134

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: TUCSON POLICE DEPT

Serial Number : 80-001199

09/19/2008

07:17:20

487
Cu
BY

Standard Lot#: 801002I

Last Changed By: S. CIONE

#50616

QAS: C. CONLEY

#34517

TUCSON PD

Operator: C. CONLEY

#34517

TUCSON PD

Subject: TEST,SQAP

DOB: 11/11/1111

Sex: M

Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	07:18:13
Diagnostic Test	Pass	07:18:40
Air Blank	0.000	07:19:08
0.100 Cal Check	0.099	07:19:27
Air Blank	0.000	07:19:56
Subject Test	0.100*	07:23:04
Air Blank	0.000	07:23:36
Air Blank	0.000	07:24:03
Subject Test	XXX**	07:24:27
Air Blank	0.000	07:25:00

*Deficient Sample

**Invalid Sample

Begin new deprivation period

Not a Successfully Completed Test Sequence

INTOXILYZER 8000

Location: TUCSON POLICE DEPT

Serial Number: 80-001199

09/19/2008

07:25:21

5507
CW

Standard Lot#: 801002I

Last Changed By: S. CIONE #50616

QAS: C. CONLEY #34517

TUCSON PD

TS4

Test	g/210L	Time
Air Blank	0.000	07:25:48
Cal Check	0.098	07:26:07
Air Blank	0.000	07:26:36

Clare Conley

Operator's Signature

EXHIBIT G-2

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY

STANDARD QUALITY ASSURANCE PROCEDURES
INTOXILYZER MODEL 8000

STANDARD CALIBRATION CHECK PROCEDURE

154 9-25-08
6587
cu

QA Specialist Clare Conley AGENCY Tucson Police Department
DATE 9-19-08 TIME 0704
Intoxilyzer Serial # 80-001199 Location Crime Lab

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.
OR
Pour a standard alcohol concentration solution _____ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 101 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Clare Conley

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000

Location: TUCSON POLICE DEPT

Serial Number: 80-001199

09/19/2008

07:04:24

Standard Lot#: 801002I

Last Changed By: S. CIONE #50616

QAS: C. CONLEY #34517

TUCSON PD

Test g/210L Time

Air Blank 0.000 07:04:51
Cal Check 0.101 07:05:10
Air Blank 0.000 07:05:40

C. Conley

Operator's Signature

7687
cu
BY