



THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

Cu  
2-8-08

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 2/6/08 TIME 13:43

INTOXILYZER SERIAL # 80-002201 LOCATION Crime Lab

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 096 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS

---



---



---



---

SIGNATURE Sarah Cione

INTOXILYZER 8000  
Location: ARIZONA  
Serial Number : 80-002201  
02/06/2008

13:44:40

2 of 10 sc

On 2-8-08

---

Standard Lot#: 719801I  
Last Changed By: S. CIONE #50616

---

QAS: S. CIONE #50616  
TUCSON PD

---

Operator: S. CIONE #50616  
TUCSON PD

---

Subject: SQAP, TEST  
DOB: 11/11/1111  
Sex: F Weight: 111

---

15 Minute Deprivation Period? Yes

---

Test	g/210L	Time
Air Blank	0.000	13:45:36
Diagnostic Test	Pass	13:46:03
Air Blank	0.000	13:46:31
0.100 Cal Check	0.097	13:46:50
Air Blank	0.000	13:47:19
Subject Test	0.000	13:47:37
Air Blank	0.000	13:48:04
Five Minute Wait	ABT*	
Air Blank	0.000	13:48:45

\*Sequence Aborted

**Not a Successfully Completed Test Sequence**

3 of 10 sc

INTOXILYZER 8000  
Location: ARIZONA  
Serial Number : 80-002201  
02/06/2008

13:48:52

Cu  
2-8-08

Standard Lot#: 719801I  
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616  
TUCSON PD

Operator: S. CIONE #50616  
TUCSON PD

Subject: SQAP, TEST  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:49:42
Diagnostic Test	Pass	13:50:09
Air Blank	RFI*	13:50:19
Air Blank	IPS**	13:50:28

\*RFI Detect  
\*\*Improper Sample

**Not a Successfully Completed Test Sequence**

4 of 10  
SC

INTOXILYZER 8000  
Location: ARIZONA  
Serial Number : 80-002201  
02/06/2008

13:50:38

lu  
2-8-08

Standard Lot#: 719801I  
Last Changed By: S. CIONE #50616

QAS: S. CIONE #50616  
TUCSON PD

Operator: S. CIONE #50616  
TUCSON PD

Subject: SQAP, TEST  
DOB: 11/11/1111  
Sex: F Weight: 111

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	13:51:33
Diagnostic Test	Pass	13:52:00
Air Blank	0.000	13:52:28
0.100 Cal Check	0.097	13:52:47
Air Blank	0.000	13:53:16
Subject Test	0.156*	13:56:25
Air Blank	0.000	13:56:57
Air Blank	0.000	13:57:25
Subject Test	XXX**	13:57:48
Air Blank	0.000	13:58:22

\*Deficient Sample

\*\*Invalid Sample

Begin new deprivation period

**Not a Successfully Completed Test Sequence**

INTOXILYZER 8000  
Location: ARIZONA  
Serial Number: 80-002201  
02/06/2006

13:56:47

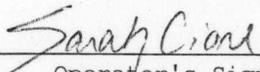
5 of 10 SC

CU  
2-8-08

-----  
Standard Lot#: 719801I  
Last Changed By: S. CIONE #50616  
-----

QAS: S. CIONE #50616  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	13:59:14
Cal Check	0.096	13:59:33
Air Blank	0.000	14:00:03

  
\_\_\_\_\_  
Operator's Signature

6 of 10  
5C

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

CM  
2-8-08

STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Sarah Cione AGENCY Tucson Police Department  
DATE 2/6/08 TIME 13:43  
Intoxilyzer Serial # 80-002201 Location Crime Lab

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 096 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

INTOXILYZER 8000  
Location: ARIZONA  
Serial Number: 80-002201  
02/06/2008 13:43:14

7 of 10  
SC

cu  
2-8-08

-----  
Standard Lot#: 719801I  
Last Changed By: S. CIONE #50616

-----  
QAS: S. CIONE #50616  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	13:43:41
Cal Check	0.096	13:44:00
Air Blank	0.000	13:44:30

Sarah Cione  
Operator's Signature



TUCSON POLICE DEPARTMENT  
CRIME LABORATORY

8 of 10  
50

INTOXILYZER SERVICE RECORD

CM  
2-8-08

QAS: Sarah Cione

Intoxilyzer Serial Number: 80-002201

Date Returned to Service: 2/6/08



TUCSON POLICE DEPARTMENT  
CRIME LABORATORY

9 of 10  
5C

INTOXILYZER SERVICE RECORD  
FOR  
CALIBRATION GAS TANK CHANGE

cu  
2-8-08

Date: 2/6/08  
QAS: Sarah Cione  
Intoxilyzer Serial Number: 80-002201  
New Lot Number: 719801 I  
New Expiration Date: July 23, 2009

- (✓) 1. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 2. Remove the old gas tank and install the new one.
- (✓) 3. Hit the "ESC" key twice and swipe the QAS card.
- (✓) 4. Go to the "M-Maintenance" menu and select "S-Standard Change."  
Select "D" for dry gas and enter the following information:  
Enter Target Value: 0.100  
Enter Control Lot #: From the tank label  
Enter Exp Date: From the tank label (mm/dd/yyyy)  
Edit info Y/N? Y or N  
Update Std Info? Y ENTER  
"Please wait saving..."
- (✓) 5. Hit the "ESC" key twice.
- (✓) 6. Go to the "M-Maintenance" menu and select "T-Control Testing."  
Select "D" for dry control test. After selection is made press ENTER.
- (✓) 7. Air blank completed.
- (✓) 8. Calibration check completed. Test results 0. 097 AC.
- (✓) 9. Air blank completed.
- (✓) 10. Remove and sign printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

INTOXILYZER 8000  
Location: ARIZONA  
Serial Number: 80-002201  
02/06/2008

13:41:10

10 of 10<sub>sc</sub>

CM 2-8-08

-----  
Standard Lot#: 719801I  
Last Changed By: S. CIONE #50616  
-----

QAS: S. CIONE #50616  
TUCSON PD  
-----

Test	g/210L	Time
Air Blank	0.000	13:41:37
Cal Check	0.097	13:41:56
Air Blank	0.000	13:42:26

*Sarah Cione*

Operator's Signature



2 of 2 sc

CU

2-8-08

# CERTIFICATE OF CALIBRATION

## **INTOXILYZER**® 8000

This is to certify that the calibration of *INTOXILYZER* serial number 80-002201, manufactured by CMI, subsidiary of MPD, Inc. of Owensboro, Kentucky, was tested and found to be in compliance with the National Safety Administration Standard for Devices to Measure Breath Alcohol (F.R., No. 179 48705-48710 Sept. 17, 1993) and with International Recommendation OIML R126 Edition 1998 (E). Calibration solutions are traceable to NIST (NBS) standard material 1828.

Date 1-23-08

Signed Del Howard  
Technician

**CMI** INC.

316 EAST 9TH STREET  
OWENSBORO, KY. 42303

Pursuant to Section 28.1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.