



CERTIFICATION by CUSTODIAN of RECORDS  
Tucson Police Department Crime Laboratory

TS4  
12-13-07

I hereby certify that these documents are a true and correct copy of the periodic maintenance\* for the Intoxilyzer 8000 Serial Number 80-002261 maintained by the Tucson Police Department Crime Laboratory pursuant to the requirements of the Arizona Department of Public Safety for a quality assurance program.

A quality assurance specialist (QAS) tested the instrument on the following dates. The acceptable accuracy limits, as defined in A.A.C. R13-10-104.A.2, for a 0.100AC solution are 0.090AC to 0.110AC.

Date: 11/19/07 Cal 1 0.100 AC Cal 2 0.099 AC

Between the dates of 10/19/07 and 11/19/07 the instrument was operating properly and accurately.

The instrument was taken out of service for repair on —.  
The instrument was returned to service on —.


Sarah L. Cione

Sarah L. Cione

11/29/07

ADPS QAS Permit # 33355

SUBSCRIBED AND SWORN to before me this 29, day of November, 2007.

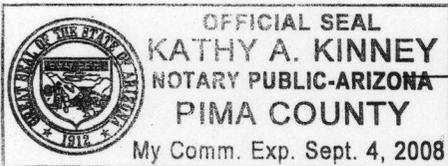

Kathy A Kinney

Notary Public

\*Includes documents specified in A.A.C. R13-10-104.A.5

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

1 of 7 SC

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000

TBY  
12.13.07

STANDARD QUALITY ASSURANCE PROCEDURE

QA SPECIALIST Sarah Cione AGENCY Tucson Police Department

DATE 11/19/07 TIME 20:52

INTOXILYZER SERIAL # 80-002261 LOCATION Vehicle # 816

(✓) 1. Display reads "PUSH BUTTON TO START"

DIAGNOSTIC TESTS

- (✓) 1. Clock time check.
- (✓) 2. Date check.

OPERATIONAL TESTS

- (✓) 1. Alcohol-free subject test result 0. 000 AC.
- (✓) 2. Error recognition logic system functioning.  
Not a Successfully Completed Test Sequence printed
- (✓) 3. Proper sample recognition system.  
Not a Successfully Completed Test Sequence printed  
Deficient sample printed.
- (✓) 4. Standard Calibration Check standard 0. 100 AC  
Result: 0. 099 AC

Instrument is operating properly and accurately. YES  NO

COMMENTS none  
\_\_\_\_\_  
\_\_\_\_\_  
\_\_\_\_\_

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

2 of 7 SC

T84

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002261  
11/19/2007 20:56:46

-----  
Standard Lot#: 7198011  
Last Changed By: T. GALLEGOS #28847  
-----

QAS: S. CIONE #50616  
TUCSON PD  
-----

Operator: S. CIONE #50616  
TUCSON PD  
-----

Subject: SQAP, TEST  
DOB: 11/11/1111  
Sex: F Weight: 111  
-----

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	20:58:05
Diagnostic Test	Pass	20:58:34
Air Blank	0.000	20:59:02
0.100 Cal Check	0.099	20:59:20
Air Blank	0.000	20:59:50
Subject Test	0.000	21:00:13
Air Blank	0.000	21:00:40
Five Minute Wait	ABT*	
Air Blank	0.000	21:02:46

\*Sequence Aborted

Not a Successfully  
Completed  
Test Sequence

3 of 7 sc

Tsh

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 8C-002261  
11/19/2007 21:05:34

-----  
Standard Lot#: 7198011  
Last Changed By: T. GALLEGOS #28847  
-----

DAS: S. CIONE #50616  
TUCSON PD  
-----

Operator: S. CIONE #50616  
TUCSON PD  
-----

Subject: SOAP, TEST  
DOB: 11/11/1111  
Sex: F Weight: 111  
-----

15 Minute Deprivation Period? Yes

-----  
Test g/210L Time  
-----  
Air Blank RFI\* 21:06:46  
Air Blank IPS\*\* 21:06:57

\*RFI Detect  
\*\*Improper Sample

Not a Successfully  
Completed  
Test Sequence

4 of 75C

PH

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002261  
11/19/2007 21:08:52

-----  
Standard Lot#: 7198011  
Last Changed By: T. GALLEGOS #28847  
-----

OAS: S. CIONE #50616  
TUCSON PD  
-----

Operator: S. CIONE #50616  
TUCSON PD  
-----

Subject: SOAP, TEST  
DOB: 11/11/1111  
Sex: F Weight: 111  
-----

15 Minute Deprivation Period? Yes

Test	g/210L	Time
Air Blank	0.000	21:10:04
Diagnostic Test	Pass	21:10:33
Air Blank	0.000	21:11:00
0.100 Cal Check	0.100	21:11:19
Air Blank	0.000	21:11:49
Subject Test	0.057*	21:14:57
Air Blank	0.000	21:15:30
Air Blank	0.000	21:15:57
Subject Test	XXX**	21:16:23
Air Blank	0.000	21:16:57

\*Deficient Sample  
\*\*Invalid Sample  
Begin new deprivation period

Not a Successfully  
Completed  
Test Sequence

5 of 7sc

TJH

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002261  
11/19/2007 21:17:59

-----  
Standard Lot#: 7198011  
Last Changed By: T. GALLEGOS #28847  
-----

OAS: S. CIONE #50616  
TUCSON PD

-----  
Test g/210L Time  
-----  
Air Blank 0.000 21:18:26  
Cal Check 0.099 21:18:45  
Air Blank 0.000 21:19:16

*Sarah Cione*  
-----  
Operator's Signature

THIS REPORT PREPARED PURSUANT TO DUTY IMPOSED BY  
A.A.C. R13-10-104(A)

6 of 75C  
154

ARIZONA DEPARTMENT OF PUBLIC SAFETY  
STANDARD QUALITY ASSURANCE PROCEDURES  
INTOXILYZER MODEL 8000  
STANDARD CALIBRATION CHECK PROCEDURE

QA Specialist Sarah Cione AGENCY Tucson Police Department  
DATE 11/19/07 TIME 20:52  
Intoxilyzer Serial # 80-002261 Location Vehicle #816

- (✓) 1. Ensure that gas tank is attached to instrument and contains a standard alcohol concentration solution 0. 100 AC.  
OR  
Pour a standard alcohol concentration solution \_\_\_\_\_ AC, into a clean dry simulator and assemble the simulator. Ensure that a tight seal has been made. Turn on the simulator and allow temperature to reach 34°C ± 0.2°C
- (✓) 2. Intoxilyzer 8000 display reads "PUSH BUTTON TO START"
- (✓) 3. Go to the "Control Testing Menu". Select "D" for dry control test or "W" for wet control test. After selection is made press ENTER.
- (✓) 4. Air blank completed.
- (✓) 5. Calibration check completed. Test results 0. 100 AC.
- (✓) 6. Air blank completed.
- (✓) 7. Remove printed record. Attach the record to the completed checklist.

SIGNATURE Sarah Cione

Pursuant to section 28-1327, Arizona Revised Statutes, this document is a certified duplicate of the information contained in computer storage devices of the Tucson Police Department.

7 of 75C

TS4

INTOXILYZER 8000  
Location: TUCSON POLICE DEPT  
Serial Number: 80-002261  
11/19/2007 20:52:30

-----  
Standard Lot#: 7198011  
Last Changed By: T. GALLEGOS #28847  
-----

QAS: S. CIONE #50616  
TUCSON PD

-----  
Test g/210L Time  
-----  
Air Blank 0.000 20:52:58  
Cal Check 0.100 20:53:17  
Air Blank 0.000 20:53:47

-----  
*Smah Cione*  
Operator's Signature  
-----